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
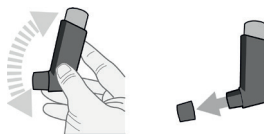
fig. 4

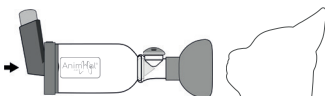Indications

The AnimHal® Cat inhalation chamber is a veterinary device. AnimHal® Cat enables and optimizes inhaled drug delivery for cats. AnimHal® Cat affects drug delivery, so it is essential to use carefully in accordance with your veterinarian's advice, and to always respect the prescribed dosage. If you have any questions or doubts, please consult your veterinarian.

Instructions for use

Before first use, it is recommended to rinse AnimHal® Cat with clean water and leave it dry.

1. Use the inspiratory valve (d) only if your cat is heavier than 9kg (if required, follow the instructions to disassemble the inhalation chamber and to re-assemble it to remove the inspiratory valve). Check the correct assembly of AnimHal® Cat (fig. 2), then, when necessary, check the correct positioning of the valve (fig. 10).

2. Check there is no foreign object in AnimHal® Cat. Shake the metered dose inhaler and remove the cap (see the instructions given with the metered dose inhaler and fig. 3). Insert the mouthpiece of the metered dose inhaler into the rear connector of AnimHal® Cat (fig. 4).

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Instructions for use (suite)

3. Position AnimHal® Cat mask over the mouth and nose of your cat and press gently to ensure the mask is correctly sealed over the face without generating any discomfort for your cat. Your cat's nose must be aligned with the mask cavity designed for this purpose.

4. Press the metered dose inhaler to release a dose of medication and hold the mask on your cat's face for about twenty seconds. Remove then the mask from your cat's face.

5. If your veterinarian prescribed several doses, wait for your cat to be compliant again before repeating the steps 3 and 4.

6. After using AnimHal® Cat, remove the metered dose inhaler and put back its cap, then store AnimHal Cat® as indicated in the "Storage" instructions.

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Precautions for use / contraindications

- AnimHal® Cat is designed for cats, it should not be used for humans (children or adults).
- It is recommended to check that none of AnimHal® Cat parts are missing (refer to the list in the « Content » section).
- AnimHal® Cat presents no known contraindications.
- AnimHal® Cat is not a toy, please ensure it is stored out of reach of unattended children.

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Important remarks

- When used under normal conditions, AnimHal® Cat must be replaced after 12 months of use. Please note that this time frame is given as an indication only, a visual inspection of the condition of the product after each use is essential. Any breakage or cracking of the rigid part or tearing of the flexible parts must result in replacing the inhalation chamber.

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- AnimHal® Cat is guaranteed free of bisphenol A, phthalates, latex, PVC and heavy metals.
- AnimHal® Cat should not be exposed to the sun for long periods of time.
- AnimHal® Cat is made in France and is fully recyclable. When it is unusable, place it in a plastic bottle container.

Disassembly

AnimHal® Cat is delivered assembled (fig. 2). To disassemble it, follow the instructions below :

- Remove the mask (g) from the valve holder (b) (fig. 5).
- Remove the rear connector (c) from the bowl shaped tank (a), separate the valve holder (b) from the bowl shaped tank (a) (fig. 6).
- Remove the inspiratory valve (d), the valve cap (f), the expiratory valve (e) from the valve holder (b) (fig. 7).

Assembly

- Insert the expiratory valve (e) into its position on the top of the valve holder (b) (fig. 8).
- Attach the valve cap (f) to the valve holder (b) (fig.9).
- To use without inspiratory valve, go to next step. To use with the inspiratory valve (for a cat heavier than 9kg), insert the inspiratory valve (d) into its position on the valve holder (b) (fig. 10).

IMPORTANT: Respect the position of the male (j; k) and female (l; m) notches.

- Attach the valve holder (b) to the bowl shaped tank (a) (fig. 11).
- Attach the rear connector (c) to the bowl shaped tank (a); ensure the alignment of the valve cap (f) with the rear notch (n). Attach the mask (g) to the valve holder (b) (fig. 12).

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Storage

- AnimHal® Cat must be stored assembled with the mask attached or not.
- AnimHal® Cat must be stored in a clean and dry place.

Cleaning and maintenance

AnimHal® Cat must be cleaned at least once a week according to the following instructions :

- Disassemble AnimHal® Cat according to the disassembly instructions, then wash them by hand in a mixture of warm clean water with washing-up liquid.
- Rinse with clean water and leave it to dry in open air. Once the components of your AnimHal® Cat are dry, reassemble it according to the assembly instructions.
- AnimHal® Cat cannot be heat sterilized. It must not be exposed to heat higher than 70°C.

For further information, please contact our customer service :

By email : info@animhal.com

By phone : +33 2 33 88 70 94

Clinical benefits

- AnimHal® Cat enables and optimizes pulmonary deposition of inhaled corticosteroids and bronchodilators commonly used in the treatment of asthma.
- AnimHal® Cat reduces the side effects associated with systemic treatments.

Side effects

- No side effects have been observed with the use of AnimHal® Cat.
- Any incident or risk of serious incident occurring in connection with the device must be notified to the manufacturer.

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OptimHal-ProtexSom SAS